Prepare la piel que rodea la herida.

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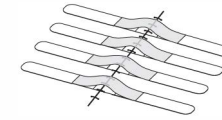
**Instructions for Use
Directives d'utilisation
Instrucciones de uso**

How does SutureSafe® work ?

SutureSafe is designed to reduce tension and pain, prevent re-opening of sutured wounds, and to close small cuts. It has a silicone elastic center that gently pulls wound edges together to support the wound. The transparent RipStop™ prevents the wound from pulling apart (especially during sudden movements such as coughing and sneezing). At all other times, the elastic center reduces tension on the wound by creating an area of relaxed skin adjacent to the incision line, reducing incisional pain and encouraging mobility.

Warnings

If an allergic reaction occurs, discontinue use. SutureSafe is for use on healthy, sutured wounds of any length, or small cuts. If the wound/cut is infected, see a doctor before use.

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For product enquiries call **Southmedic Inc.:**
1-800-463-7146


To learn more about SutureSafe®, visit:
www.southmedic.com

DYNAMIC TISSUE SYSTEMS™

Another Southmedic® Solution

| Available sizes | REF |
|--|-------|
| SutureSafe®, 1/2 in x 4 1/2 (13 mm x 114 mm), 3 pouches of 10 | DWS12 |
| SutureSafe®, 3/4 in x 8 1/2 in (19 mm x 216 mm), 3 pouches of 10 | DWS14 |

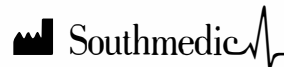50 Alliance Blvd.
Barrie, ON
Canada L4M 5K3
www.southmedic.com
1-800-463-7146

U.S. Distribution Center
111 Harvester Chicago, IL
USA 60185


EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

IFU0251E

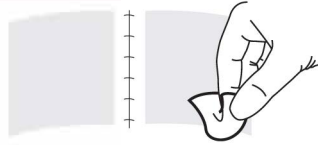1

How to apply SutureSafe®

Prepare skin around the wound.

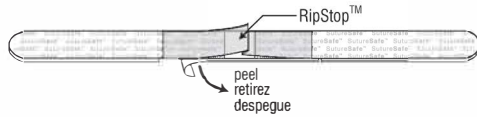
SutureSafe sticks best to clean, hairless skin. Wash skin with soap and water. Ensure skin is dry. Wipe skin with rubbing alcohol to improve adhesion. If necessary, remove body hair by shaving.

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2

Peel off bottom center liner.

Note: Avoid touching the exposed surface.

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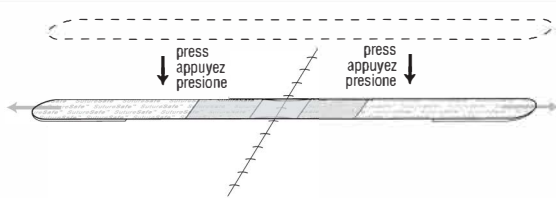
3

Gently stretch until RipStop™ becomes just flat.

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4

Center over wound, then press on skin.

Center SutureSafe as shown and press to skin on both sides at the same time.

Note: For difficult to reach areas, ask for help in applying.

Warning: Do not pull the skin as this can cause skin damage or failure of SutureSafe to stick to skin.

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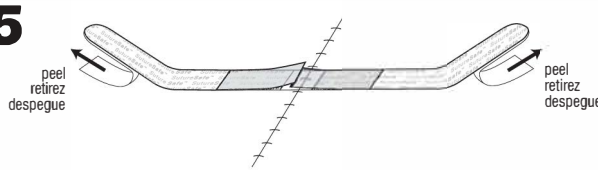
5

Peel off outer liners and apply.

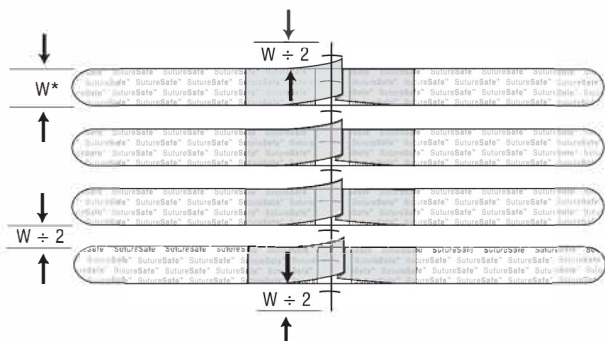
Rub fabric down gently to ensure SutureSafe sticks well to skin.

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Sutured wound / Plaie suturée / Herida suturada

Repeat steps 2 - 5 to cover wound or cut.

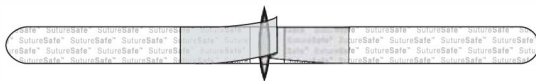
Place SutureSafe along entire length of the wound or cut, as shown. Maximum spacing between strips should be half the width of the product, as shown.

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Small cut / Petite coupure / Cortadura pequeña

Using SutureSafe®

If necessary, dressings can be applied on top of SutureSafe. SutureSafe is shower friendly. Let SutureSafe air dry prior to covering with clothing or dressings.

When to replace

Apply fresh SutureSafe as needed. Each SutureSafe can last up to seven days.

How to remove

Peel ends back toward the wound for painless removal.

Note: A sticky residue may be left on skin if SutureSafe is removed wet. To remove residue from skin, let dry, then rub off.

When to Stop

SutureSafe is recommended for use over sutures/staples/glue for a minimum of two weeks. If you are using SutureSafe to close a small cut, apply until cut heals (usually about 10 days).

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