

ABRO™ Abdominal External

Early primary fascial closure provides a better outcome

Indications for Use:

- Support and stabilize the abdominal wall
- Prevent loss of domain
- Prevent abdominal fascia damage
- Prevent lateral retraction of abdominal wall fascia
- Facilitate primary closure of abdominal wall fascia

Open Abdominal Wounds

ABRO™ Abdominal “EXTERNAL” System facilitates early fascial closure of mid-line abdominal defects.

Application of the ABRO™ System prevents lateralization of the fascial margin by engaging the abdominal wall muscles and applying a gentle, unrelenting dynamic appositional force. ABRO™ gradually reapproximates the fascial edges preventing damage to the skin and abdominal wall fascia. It is beneficial for primary fascial closure.




Closure of the abdominal wall fascia restores normal physiology, which in certain cases has been shown to reduce length of stay, short term morbidity risks and future healthcare costs.



ABRO™
Abdominal Closure System

Product	Code/Quantity
ABRO™ Abdominal External: 2x Rectus Muscle Splint (RMS) 1x Circumferential Dynamic Retainer (CDR)	CWK10

Results at a Glance

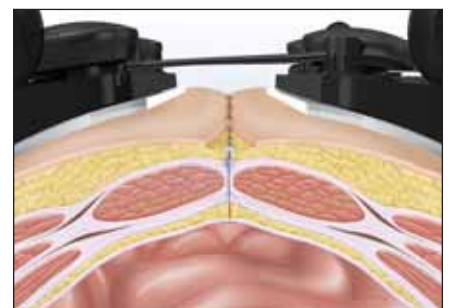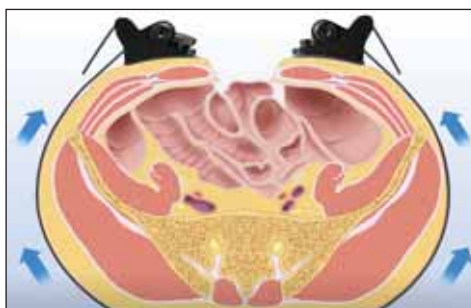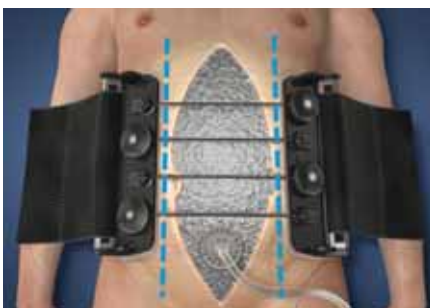

A Dynamic Tissue Systems® Solution


Open wounds impose serious clinical consequences

Dynamic Action of ABRO™

Gentle, unrelenting dynamic appositional forces (cyclic stretching) counters the retracting forces on the abdominal wall fascia that interfere with primary fascial closure.

How? Cyclic stretching of tissue decreases muscle atrophy and facilitates reapproximation of the abdominal wall fascia.

Unlike static devices, dynamic therapeutic tension rapidly addresses the challenge of the open abdomen. ABRO's therapeutic tension addresses the inertia required to return wound edges back to their original position for definitive primary closure.

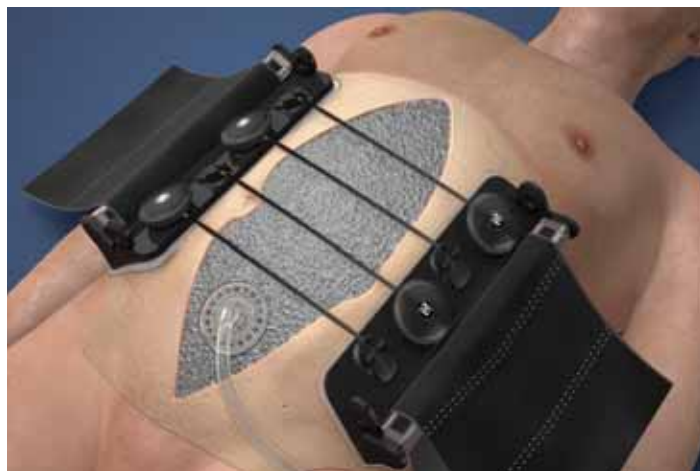

Retraction
Prevention

Dynamic Wound
Closure

Tissue Support
and Expansion



www.dynamictissuesystems.com

contactus@southmedic.com

Southmedic Inc., 50 Alliance Blvd., Barrie, ON Canada L4M 5K3 705-726-9383 1-800-463-7146 Toll free in North America

PO213 June 2017